

Greetings to our wonderful Waggoner CCVT customers. We would like to share that our latest update has been released to improve security and testing controls. To learn how you can update your Windows version of Waggoner CCVT, please click [HERE](#). Once completed, you should see v4.05 in the top left of the Home screen. For those who are using Apple devices, please check in the App Store to see if your current Waggoner app needs updating.

1. Add a signature to your patient's results page

To ensure that a color vision test was done by a licensed practitioner, there is now an option to add a signature. After completing the update, open your Settings menu and review images found [HERE](#) to see where to turn on signatures. Please note, this feature will apply to all tests administered. Also, the signature will use the information set up in the profile. A video that shows how to edit your profile if necessary is [HERE](#).

2. Monitor Settings for best testing conditions

There are a large number and variety of monitors and device types available that can run the Waggoner CCVT. In order to make sure ideal settings are utilized, you will notice a new pop up when starting the Waggoner CCVT. To see what the process for checking the settings directed looks like for iPad click [HERE](#) and Windows click [HERE](#).

3. Null & Void tests

At Waggoner, we take testing integrity very seriously. This software upgrade includes a feature to help the administrator feel confident in the testing results even if they aren't the one monitoring the patient. Due to the possibility that screen alterations could affect the test, the CCVT will now force the user to restart if it detects alterations during the test. If this happens twice in a row, the administrator will be notified at the end and given the option to declare results Null & Void or Accept. Please note, we have discovered that if a user has auto-brightness settings turned on, it could trigger the software to detect a brightness level change. To see what these notifications look like, click [HERE](#). For an example of how to correctly set auto-brightness for most Windows devices, click [HERE](#). To see where the auto-brightness adjustment is for iPads, click [HERE](#).

We are very grateful for your business and commitment to keeping our skies safe. Please reach out to sales@waggonerdiagnostics.com if you have further questions.

Thanks,

The Waggoner Diagnostics team